



PHOTON 4 HEALTH
NEXT LEVEL LIGHT THERAPY



[PHOTON4HEALTH.COM](https://photon4health.com)



[INFO@PHOTON4HEALTH.COM](mailto:info@photon4health.com)



+31 611049629



THE NETHERLANDS

REDPOWER MINI ATP QUICK START GUIDE



**IMPROVE HEALTH, INCREASE
ENERGY, REDUCE PAIN**
– ANYWHERE, ANYTIME –

Product photographs




How To Use

Congratulations on the purchase of your REDPOWER MINI next level laser photon therapy device. The REDPOWER MINI is the most powerful laser photon therapy handheld device in the world and is the result of 10 years of research and experience. The RED POWER MINI will contribute enormously to your overall health.

Methods of operation

1. By 2 sec holding the power button the device will be turned on. By short pressing it again, the treatment starts. After 5 minutes, the device will be turned off automatically.
2. If the treatment is running, you can pause the treatment by briefly pressing the power button.
3. By 2 sec holding the power button at any given time, the device will be turned off.
4. When the device is powered off, if the battery icon starts flashing you need to connect the charger. It cannot be used when being charged.

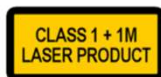

Therapeutic effect

The low-intensity infrared lasers emit exactly the type of light that your cells can easily absorb and convert into biophotons. Biophotons are used by your cells for internal communication processes and communication with surrounding cells. All your cells, muscles and organs need biophotons for proper functioning.

Unfortunately, we absorb too few biophotons through natural pathways, causing shortages. The REDPOWER MINI will help you solve this shortage. Increasing biophotons improves your self-healing ability, reduces pain, inflammation and injuries. Treat yourself for 15 minutes a day to maintain your biophoton levels in your body.

We recommend that you place the REDPOWER MINI for 5-15 minutes on the locations on your body where the biophotons are needed most. But fear not, wherever you place it they will travel throughout the body. For example, you can also use the supplied holder with Velcro straps to wear the REDPOWER MINI as a belt. If you do not have a specific treatment location in mind, place it on your heart, belly or your lower-back.

TECHNICAL PARAMETERS

ITEM	DESCRIPTION
Product name	PHOTON 4 HEALTH REDPOWER MINI ATP
Laser medium	GaA/As Semiconductor laser
Light diode	22*785nm laser @5mW 2*405nm laser @5mW 2*520nm laser @5mW 2*635nm laser @5mW 4*630nm led @50mW 4*810nm led @50mW
Led power	Total 400 mW
Laser power	Total 140 mW
Treatment	15 Minutes
Laser Class	Class 1+1M
Product size	8.0cm x 8.0cm x 1.8cm 3,2 inch x 3,2 inch x 0,7 inch

DISCLAIMERS

Warranty. Your REDPOWER MINI ATP is warranted against defects in materials and workmanship for a period of 3 YEARS from the purchase date with free repair or replacement. Warranty does not cover the follow damage(s): product been modified, improper installation or use, improper cleaning, electrical surges, or nature caused damage. Warranty does not cover third party direct or indirect damage(s).

Medical Disclaimer. The PHOTON 4 HEALTH REDPOWER MINI ATP and the information in this quick start guide is not intended as a substitute for services or information from trained medical professionals and/or healthcare providers, such as general practitioners, physicians, medical specialists, emergency care, (acute) mental health care, physiotherapists and psychotherapists. The effect of this product and other resources and/or claims, tips and advice mentioned by us are not necessarily scientifically proven. All (medical) claims and health benefits we make are based solely on our personal experience with our clients. Neither the information provided nor the options for use can be seen as a substitute for medical or other professional help, care, support or information provision and are not intended as an aid in making a medical (self) diagnosis. The information is intended as general information and is not specifically aimed at one individual person or one specific medical situation of, for example, the reader of this brochure or someone around him. This means that readers cannot attach any diagnostic or therapeutic value to the information for their own medical situation or that of others. In cases of diseases, medical complaints, treatments or research, the information is only intended as support. In such situations, the information on/in our website/brochure/videos/books/images/webinars can serve to support the contact between the reader (or their family member or other loved ones) and their treating doctor or other professional care provider and the advice provided he/she receives from the latter. Readers are advised to submit medical questions, complaints or symptoms in a timely manner to the treating physician or other healthcare professional in order to receive professional examination, diagnosis and possible medical care. Readers are advised never to delay seeking timely professional care or to never neglect medical advice already obtained based on (online) information. In case of acute medical complaints or psychological problems and if complaints or symptoms worsen, readers are advised to contact their own doctor or the emergency number.