



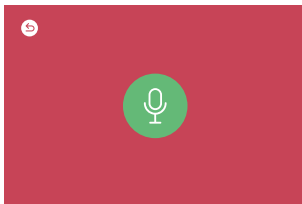

Voice Control

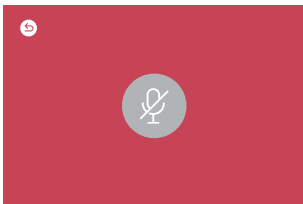

Turn on/off the voice control function

Start with : Hello RED POWER / Hi RED POWER

followed by:

- Run No Pulse mode
- Run DNA Repair mode
- Run Custom mode
- Run Delta Sleep mode
- Run Theta Dream mode
- Run Slow Alpha mode
- Run Basic Alpha mode
- Run Fast Beta mode
- Run Active Gamma mode
- Volume up/down
- Turn on/off Bluetooth
- Stop

Warranty clause

- 1) REDPOWER PRO XL medical series are warranted against defects in materials and workmanship for a period of three (3) YEARS from the purchase date with free repair or replacement.
- 2) Warranty does not cover the follow damage(s): product been modified, improper installation or use, improper cleaning, electrical surges, or nature caused damage.
- 3) Warranty does not cover third party direct or indirect damage(s).



PHOTON 4 HEALTH
NEXT LEVEL LIGHT THERAPY



PHOTON4HEALTH.COM




INFO@PHOTON4HEALTH.COM



+31 611049629




THE NETHERLANDS

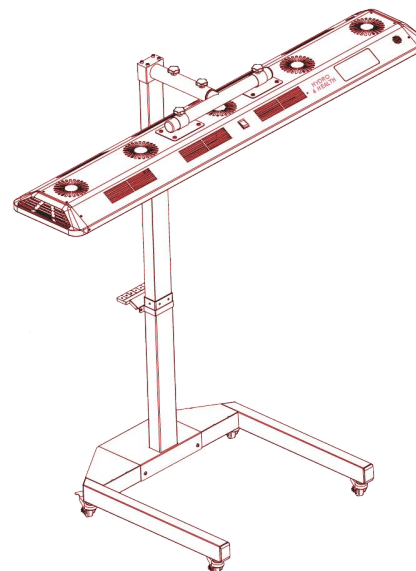


Medical Disclaimer

The REDPOWER PRO XL and the information in this quick start guide is not intended as a substitute for services or information from trained medical professionals and/or healthcare providers, such as general practitioners, physicians, medical specialists, emergency care, (acute) mental health care, physiotherapists and psychotherapists. The effect of this product and other resources and/or claims, tips and advice mentioned by us are not necessarily scientifically proven. All (medical) claims and health benefits we make are based solely on our personal experience with our clients. Neither the information provided nor the options for use can be seen as a substitute for medical or other professional help, care, support or information provision and are not intended as an aid in making a medical (self) diagnosis. The information is intended as general information and is not specifically aimed at one individual person or one specific medical situation of, for example, the reader of this brochure or someone around him. This means that readers cannot attach any diagnostic or therapeutic value to the information for their own medical situation or that of others. In cases of diseases, medical complaints, treatments or research, the information is only intended as support. In such situations, the information on/in our website/brochure/videos/books/images/webinars can serve to support the contact between the reader (or their family member or other loved ones) and their treating doctor or other professional care provider and the advice provided he/she receives from the latter. Readers are advised to submit medical questions, complaints or symptoms in a timely manner to the treating physician or other healthcare professional in order to receive professional examination, diagnosis and possible medical care. Readers are advised never to delay seeking timely professional care or to never neglect medical advice already obtained based on (online) information. In case of acute medical complaints or psychological problems and if complaints or symptoms worsen, readers are advised to contact their own doctor or the emergency number.

QUICK START GUIDE

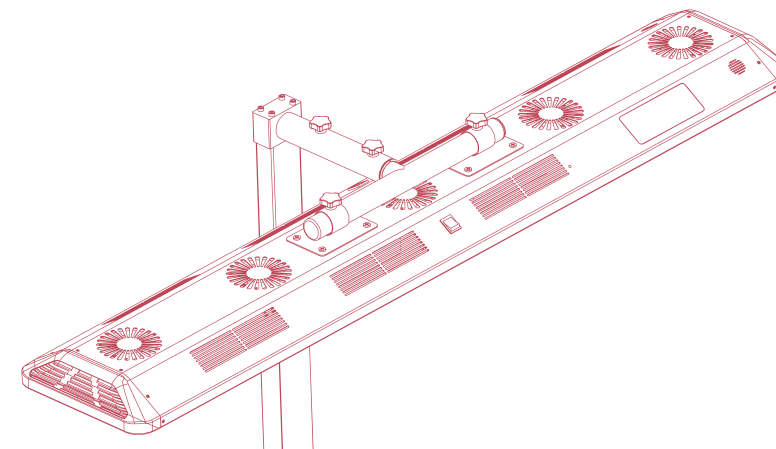

PHOTON 4 HEALTH
NEXT LEVEL LIGHT THERAPY

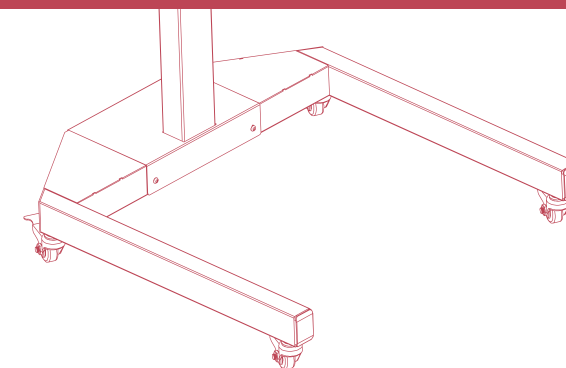

REDPOWER PRO XL

Congratulations on the purchase of your REDPOWER XL next level photon therapy lamp. The REDPOWER PRO XL is the most powerful red light therapy lamp in the world and is the result of 10 years of research and experience with over 10.000 patients. It combines (infra)red light therapy with Low Level Laser Therapy, Laser ATP Therapy and Tesla magnetic field therapy. This product will greatly contribute to your and/or your patients overall health.



9 different modes

NO PULSE
Unpulsed

DNA REPAIR
528Hz

CUSTOM
Customized Values

DELTA SLEEP
Sleep improvement

THETA DREAM
Dream Stimulation

SLOW ALPHA
Deep relaxation

BASIC ALPHA
Relaxation

FAST BETA
Learning/Creativity

ACTIVE GAMMA
Dementia/Alzheimer's

Swipe for page




Start treatment


Turn on the power, the device will boot up and will show the home screen where you will see 9 different modes. 8 pre-programmed modes + 1 custom mode that you can set to your own wishes. (Changing light intensity, treatment duration & pulsation of the infrared LEDs).

The pre-programmed modes each have a treatment time of 15 minutes. Click on the icon of the mode you want to start. Or start the program using the built-in voice control (see later in this manual). (Optimal distance from lamp to body is 15cm - 20cm / 6" - 8").

We recommend using the device for 15 minutes every day. Longer is allowed, but has no added value. With the exception of any additional treatments AFTER heavy physical exertion such as sports to shorten recovery time and reduce injuries. Preferably you treat yourself in the morning. Treat the front AND back of your body in 15 minutes. Treat your face with the side where the white plate/ screen is located. The light on this side is safe for your eyes.

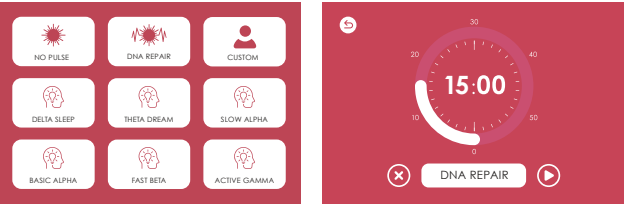

Turn on Bluetooth, find device "REDPOWER" in your phone, connect to play music!

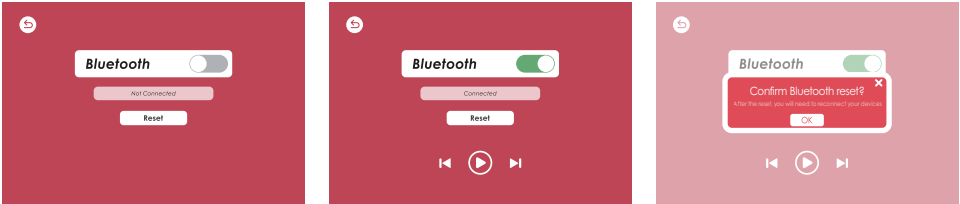

Technical specifications

PHOTON 4 HEALTH
REDPOWER PRO XL

384 pcs high power medical grade leds
176 pcs medical grade 6mW lasers
Max. 600 Watt power consumption

4 laser wavelength for maximum results
450nm + 520nm
650nm + 785nm

7 led wavelength for maximum results
630nm + 660nm
785nm + 810nm
830nm + 850nm + 940nm

Magnetic field therapy
Built-in Tesla-coil Schumann & Alpha
resonance magnetic field therapy

Dimensions light unit
164 * 30 * 7,5 cm / 65" * 12" * 3"

Weight
Total weight 50 kg / 111 lbs
Light unit = 23 kg / 51,5 lbs, stand 27kg / 59,5 lbs

Safety measures
Short circuit protection
Overcurrent protection
Overload protection
Overheat protection

